

Enbio S

Class B Autoclave

*This Quick Reference Guide was created by DentalMarketplace as a general overview. It is **not** the official Enbio manual, which ships in the box with your unit. Always follow the official Enbio user manual, technical passport, and validation report included with your device for complete setup, safety, and maintenance instructions.*

What's in the Box

- Enbio S (medical Class B) sterilizer
- Technical passport
- Safe Studio Certificate
- Water and condensate connection hoses
- HEPA filter
- Magic filter
- User manual
- Warranty terms
- FDA certificate confirmation
- Validation report
- USB flash drive
- Plug-in cable

Cycle Selection Overview

Cycle	Time / Temp	Typical Use
FAST (unwrapped)	7 min, 273°F (134°C)	Immediate-use instruments
Standard Class B	15 min, 273°F (134°C)	Wrapped instruments, routine turnover
Heat-sensitive	45 min, 250°F (121°C)	Heat-sensitive wrapped instruments

Basic Setup

- 1 Place the Enbio S on a stable, level countertop with adequate ventilation clearance.
- 2 Connect the external water and condensate connection hoses — no permanent plumbing required.
- 3 Install the HEPA and Magic filters as shown in the official manual.
- 4 Plug in using the included power cable.
- 5 Insert the USB flash drive to enable automatic cycle data logging.

Basic Operation

- 1 Load instruments into the chamber — up to 8 sterilization pouches per cycle.
- 2 Select the appropriate cycle (FAST, Standard Class B, or Heat-sensitive) for your load.
- 3 Start the cycle and allow it to complete fully before opening the chamber.
- 4 Cycle data is automatically saved to the USB flash drive for compliance recordkeeping.
- 5 Remove sterilized instruments once the cycle indicator confirms completion.

Care & Maintenance

- Use only distilled or demineralized water in the system.
- Replace/clean the HEPA and Magic filters per the official manual's maintenance schedule.
- Keep the technical passport and validation report on file for compliance audits.
- Contact an authorized Enbio service partner for maintenance beyond routine filter care.

This Quick Reference Guide is a DentalMarketplace-created resource and does not replace the official Enbio manual included with your unit.
DentalMarketplace | (346) 885-6521 | dentalmarketplace.com