
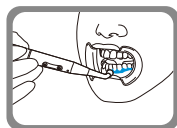


**beyond** BlueSeal<sup>®</sup>  
Gingival Protection Kit

BY-PD02205  
BY-PD02202

## Instruction Manual

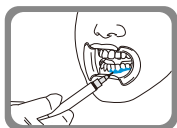

### DESCRIPTION & INDICATIONS:

The BEYOND<sup>®</sup> BlueSeal<sup>®</sup> is a sealing light-cured resin designed to work well with embrasures and undercuts for effective coverage of oral tissues. The BlueSeal<sup>®</sup>'s unique formula was created to have enough strength to maintain a strong barrier and still be easy to remove. The distinctive blue hue shows the contrast easily against the teeth and gingiva for more precise coverage.

### COMPOSITION:

Contains difunctional methacrylate oligomer (exothane 8), polyethylene glycol 400 dimethacrylate, hema phosphate, silicon dioxide, camphorquinone.

### NET WEIGHT:

Depending on kit purchased. Please refer to outside of box.

### TREATMENT PROCEDURE:

1. Air dry enamel prior to treatment.
2. Attach preferred tip to syringe body (metal tip for more exacting application, plastic tip with larger aperture for quicker application). Extrude on disposable cloth to prime before application.
3. Apply a barrier of BlueSeal 3–5mm wide on the gingiva. Make sure to seal interproximal spaces.
4. Apply resin approximately 0.5mm onto dry enamel to seal. Extend resin one tooth beyond last tooth whitened for effective coverage.
5. Light cure using a smooth, circular motion for 20-30 seconds. (based on curing light with 800-1100mW/cm<sup>2</sup> range)

6. Pull out cured resin in one piece or in a few large pieces.
7. Check for retained resin.  
BlueSeal was specifically created to remove easily following treatment.

### PRECAUTIONS:

1. For professional in-office use only.
2. Patient must wear protective eyewear.
3. Operators must wear gloves, mask and protective eyewear.
4. Do not use on pregnant or lactating women.
5. Do not use on children under 16 years of age.
6. Do not use gingival barrier on any persons having known resin allergies.
7. Anyone with a history of chemical allergies is advised to undergo allergy checks before treatment.

### FIRST AID:

- SKIN/TISSUES (contact): Immediately wash thoroughly with water. Seek medical attention if symptoms persist.
- EYES (contact): Open the eye wide and thoroughly wash for 15 minutes with running water. Seek medical attention if symptoms persist.
- INGESTION: Rinse mouth with water. Gargle with salt water and drink lots of milk or water. Seek medical attention if symptoms persist.
- INHALATION (powder): Remove to fresh air. If breathing is difficult, seek medical attention.

### STORAGE AND HANDLING:

- Avoid exposure to heat or sunlight. Store in room temperature.
- Do not use after expiration date.

### PACKAGE CONTENTS:

BEYOND<sup>®</sup> BlueSeal<sup>®</sup> Gingival Protection Kit 5-Pack  
5 x Syringes of BEYOND<sup>®</sup> BlueSeal<sup>®</sup> Gingival Protection (2g/1.6mL)  
5 x Metal Tips  
5 x Plastic Tips  
1 Instruction Manual

BEYOND<sup>®</sup> BlueSeal<sup>®</sup> Gingival Protection Kit Twin-Pack  
2 x Syringes of BEYOND<sup>®</sup> BlueSeal<sup>®</sup> Gingival Protection (2g/1.6mL)  
2 x Metal Tips  
2 x Plastic Tips  
1 Instruction Manual